
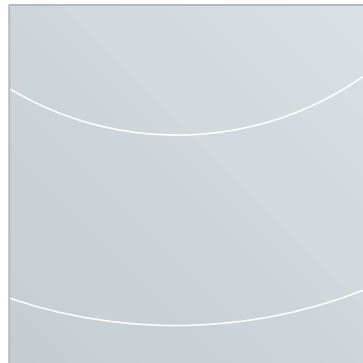


# PRESSURECARE

Pressure Relief & Patient Positioning

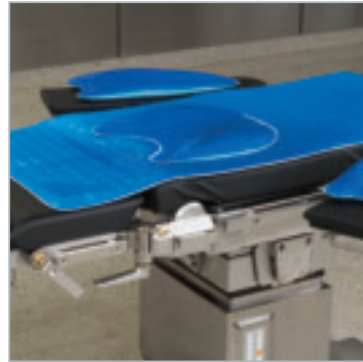

# Trulife Overview

Trulife is an internationally based and managed group engaged in the creation, development, manufacture and marketing of niche healthcare products. In addition to Pressurecare pads for the Operating Tables, the Group activities encompass product ranges, Orthopaedics, Prosthetics, Breastcare and Wheelchair cushions. For further details on all our product ranges please visit our website: [www.trulife.com](http://www.trulife.com)



Trulife products are distributed by more than 300 distributor companies worldwide. Our primary markets are the USA, Canada, the UK and Ireland, where we have our own dedicated marketing organisations.

The Group is headquartered in Dublin, Ireland. European manufacturing, research & development and marketing operations are conducted through wholly owned subsidiaries in Ireland and England. Orthopaedic clinical services are provided throughout England, Scotland and Wales.

In the Americas, Trulife trades through wholly owned subsidiary companies. Customer service and distribution facilities are based in Jackson, Michigan, Poulso, Washington and Trenton, Ontario. Manufacturing and research & development of Orthopaedics, Prosthetics and Custom Breastcare products is conducted in Poulso, Washington, Jackson, Michigan and Trenton, Ontario.

## **Mission Statement**

The core philosophy is a belief in serving customers' needs through excellence in product innovation, quality in everything we do, extensive education and training offerings, and development of patient care through committed and dedicated people.

INTRODUCTION	2
Pressurecare	2
Trulife Ranges	3
PRODUCT GUIDE	4
Table Pad Protection	4
Head & Neck Protection	7
Arm & Elbow Protection	10
Body Protection	12
Leg & Knee Protection	17
Ankle, Heel & Foot Protection	18
Speciality Pad Protection	20
Paediatric Products	21
Accessories	25
POSITIONING GUIDE	26
Supine	26
Prone	27
Lateral	28
Lithotomy	29
TECHNICAL INFORMATION	30
Interface Pressure Maps	30
General Use	32
Index	33

[1] "In the UK Pressure Sores absorb around 4% of the National Healthcare expenditure"

[2] "2.5 million patients are treated for pressure sores in the USA"

## Pressurecare Prevention of Peri-Operative Pressure Injury

### Definition: Pressure Ulcer

'A pressure ulcer is localized injury to the skin and/or underlying tissue usually over a bony prominence, as a result of pressure, or pressure in combination with shear'  
NPUAP / EPUAP 2011 [3]

### Definition: Pressure Neuropathy

'In pressure neuropathy there is excessive pressure on a nerve root or a peripheral nerve leading to ischemia (not enough oxygen due to poor blood supply).. If the pressure continues, fibrosis tends to develop.' [4]

### Who is at Risk?

All Operating Room (OR) patients due to level of Impaired Mobility. Surgical patients still account for 20% to 30% of all pressure injuries.[5]  
General Statistics on Prevalence of PUs  
Germany (21.1%) [6], Australia (28.2%) [7], Italy (27%) [8]  
US ( 11.5% ) [9], Canada ( 16.5% ) [9].

### Areas Most at Risk

Bony Prominences – See Diagram  
Head : Elbows : Sacrum : Heels : Shoulders : Hips : Buttocks : Knees.

### Aetiology: What happens?

Prolonged Pressure ▶ Pressure on Blood Capillaries / Nerves  
▶ Ischaemia & lack of Oxygen ▶ Skin Breakdown or Nerve Compression  
▶ Pressure Ulcer / Neuropathy


### Principles of Safe Positioning

▼ Level of prolonged pressure on bony prominences and nerve endings  
▶ Avoid Circulatory and Respiratory Insufficiency  
▶ Attain Optimal Surgical Position Safely  
▶ Facilitate Role of Anaesthetist

## Intervention - Pressure Injuries can be prevented! Use Preventative measures! Educate Staff!

### Trulife offers an Extensive Range of Equipment for Pressurecare Management in the OR

The prevention of pressure injury assumes a fundamental role in health services economics and management. [10]

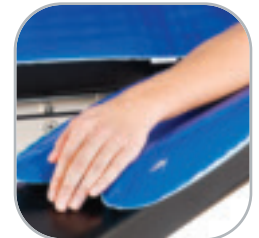

[1] Preventing Pressure Ulcers: The Goal Is Zero K. Duncan, R.N. The Joint Commission Journal on Quality and Patient Safety 2007; Vol 33 ( 10 ) 605 – 610

[2] The Cost of pressure ulcers in the UK G. Bennett, C. Dealey, J. Posnett age and ageing: Vol 33 ( 3 ) 230 – 235

[3] www.epuap.org www.npuap.org [4] www.medterms.com

[5] Patient Positioning: Education is key. www.surgicalproductsmag.com. 2010. Claire Williams Tager

[6] Lahmann N. et al., J. Ostomy/Wound Management.52: 20-33, 2006. [7] Gardner A. et al., J. Wound Practice and Research. 17:134-145, 2009.

[7] Lahmann N. et al., J. Ostomy/Wound Management.52: 20-33, 2006. [8] Capon A. et al., J. Advanced Nursing. 58: 263-272, 2007.

[9] Healthcare Management Council Inc. HMC: Hospitals lose \$2 Million in errors yearly. March 5, 2010. Assessed March 11,

2010 from: <http://www.hmc-benchmarks.com/2010/03/hmchospitals-lose-2-million-in-errors-yearly/>

[10] ECONOMIC EVALUATION OF PREVENTION OF PRESSURE ULCERS IN INTENSIVE CARE UNITS

Andrade, Melanie 1\*, Nogueira, Fernanada 2, Morais, Ernesto 3: EPUAP Annual Meeting 2011

The incidence of OR-acquired pressure sores can be significantly reduced by using the most effective pressure relieving and positioning devices available and by employing a few basic concepts in positioning care. Education of OR staff is paramount.

The Trulife Pressurecare Ranges were developed as a result of our commitment to provide improved levels of pressure care products to both patients and healthcare systems. Vast amounts of in-depth research have gone into the development of each range. The Trulife Pressurecare range offers a simple and cost effective solution with a comprehensive offering to cover all surgical procedures.

**Why select the Trulife Range?**

- Extensive range
- Different budgets considered
- Medical Grade Silicone
- Latex Free
- MRI Compatible
- X-Ray Translucent
- Biocompatible
- Reusable
- Cost Saving
- Made in Ireland



**OASIS - THE ORIGINAL SILICONE SOLUTION**

The Oasis range is an extensive range of reusable 100% silicone gel pads. The silicone we use is softer than both skin and underlying tissue. This means that pressure is relieved by the slight movement of the silicone gel and resulting dissipation of pressure/force across the product. It is the first range of products launched by Trulife in the OR field and remains a market leader in gel pads and positioners.



**OASIS ELITE - LIGHTWEIGHT & PRE CONTOURED**

The Oasis Elite range combines silicone gel and light weight foam with a precontoured surface to provide superior pressure relief and patient protection in the OR. Anatomical pre-contouring facilitates greater conformity between gel and body shape; weight is then redistributed over a larger area. Oasis Elite products are more than 50% lighter than comparable gel-only products. They are designed for stability, easier handling and manipulation by medical personnel during preparation and surgery.



**AZURE - REDEFINING PRESSURECARE**

The Azure range is the most advanced and innovative range of products available in the OR today. The Azure range offers the best pressure redistribution. Each product consists of two independent protective silicone layers that work in combination allowing the moulded products to effectively conform to the individual user's shape in a more contoured and less structured manner, resulting in effective protection against nerve damage and skin breakdown. The new approach by Trulife was to create a product that replicates and acts as an additional layer of protective tissue and skin. Azure has set a whole new standard of protection in the OR.

**Surgical Positions products are suitable for:**

Supine **S**      Prone **P**      Lithotomy **L**      Lateral **LL**      Paediatrics **P**      \* Indicates use



S P L LL



P

### AZURE PROFILE PADS

The Azure Table Pads combine the ultra-soft and lightweight silicone material on the base and the cured silicone gel on top, with a unique squared profile that cushions the body during surgery. The pads are up to 20% lighter than comparative gel pads.

CODE	DESCRIPTION	DIMENSIONS
AZ610	Universal Table Pad	520 x 520 x 10mm
AZ611	¾ Length Table Pad	1150 x 520 x 10mm

S P L LL



P

### AZURE TABLE PADS

The Azure Table pads can be used to protect the body in all surgical positions. The Azure Table Pads combine the ultra - soft and lightweight silicone material of the base and the cured silicone gel on top. The pads are up to 20% lighter than comparative gels.

CODE	DESCRIPTION	DIMENSIONS
AZ600	Operating Table Pad	520 x 520 x 10mm
AZ601	Operating Table Pad	1150 x 520 x 10mm

S P L LL



P

### OASIS ELITE TABLE PADS

The Oasis Elite Pads can be used to protect the body in all surgical positions. The ultra light weight pad consists of a foam base and a gel top, allowing for easy carrying and manipulation by OR staff.

CODE	DESCRIPTION	DIMENSIONS
EL030	Universal Table Pad	520 x 520 x 30mm
EL031	¾ Length Table Pad	1150 x 520 x 20mm

S P L LL



P


### OASIS STANDARD OPERATING TABLE PADS

These Oasis table pads provide protection over the entire surface area of the operating table and are recommended for use with patients in all positions. The pads are flexible, soft, comfortable and versatile and distribute weight evenly over the entire surface area.

CODE	DESCRIPTION	DIMENSIONS
OA030	Universal Table Pad	520 x 520 x 10mm
OA031	¾ Length Table Pad	1150 x 520 x 10mm
OA032	Full Length Table Pad	1800 x 520 x 10mm



S P L LL

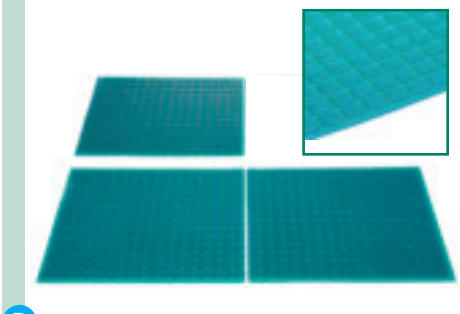

P

### OASIS PROFILE PAD

The Oasis Profile Pad has a series of 30mm<sup>2</sup> raised Trugel profiles that increase the overall surface area of the pad by 33%. This unique design allows for greater weight distribution and is particularly effective during extended procedures.

CODE	DESCRIPTION	DIMENSIONS
OA041	Universal Profile Pad	520 x 520 x 12mm
OA042	3/4 Length Profile Pad	1145 x 520 x 12mm

S P L LL



P

### OASIS 3 PIECE OPERATING TABLE PAD

The Oasis 3 Piece Operating Table Pad is designed for use on sectioned tables. The set consists of a head section, torso section and a foot section, with the latter two having a perineal cutout. The product provides a high degree of flexibility, versatility and protection during a variety of surgical procedures, particularly in the areas of Orthopaedics and Urology. Can also be used as a standard full length table pad.

CODE	DESCRIPTION	DIMENSIONS
OA036	Head Section	225 x 500 x 10mm
OA037	Torso Section	990 x 500 x 10mm
OA038	Foot Section	500 x 500 x 10mm
OA039	3 Piece Set	-



P

### OASIS FULL LENGTH PAEDIATRIC PAD

Used to protect paediatric patients during all surgical procedures, this pad can also be used as a mattress overlay in hospital beds.

CODE	DESCRIPTION	DIMENSIONS
OA033	Paediatric Full Length Pad	605 x 335 x 10mm

S P L LL

**OASIS HIP PAD**

The Oasis Hip Pad is a flat pad specifically designed for use on hip fracture table tops.

CODE	DESCRIPTION	DIMENSIONS
OA035	Hip Pad	520 x 520 x 10mm

P

S P L LL

**OASIS PERINEAL TABLE PAD**

The Oasis Perineal Pad provides excellent protection to the sacral area, a point of maximum pressure. It is essential for all patients in the lithotomy position.

CODE	DESCRIPTION	DIMENSIONS
OA040	Perineal Table Pad	520 x 520 x 10mm

P

**INCUBATOR PAD**

This unique ultra soft foam pad provides a soft and comforting base for neonates while providing all-round protection from pressure sores and nerve damage.

CODE	DESCRIPTION	DIMENSIONS
OA300	Incubator Pad	620 x 340 x 12mm

P




S L LL



**AZURE HEAD PROTECTOR**

The dual layer Azure Head Protector is innovatively designed to contour to the individual patient's head, conforming comfortably while providing effective protection to the patient.

CODE	DESCRIPTION	DIMENSIONS
*AZ100	Infant/Adolescent/Adult	260 x 250 x 50mm

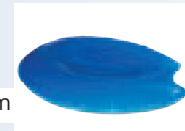

P \* Infant/Adolescent only

P



**OASIS ELITE PRONE HEAD REST**

Developed in conjunction with the Operating Staff at a leading hospital, the Oasis Elite Prone Head Rest is anatomically designed so that the patient's head can comfortably rest face down with respiratory tubes in place. The unique design provides excellent support, stability, comfort and protection during surgery in the prone position, with openings on both sides. The product is lightweight, durable and easy to handle.

CODE	DESCRIPTION	DIMENSIONS
*EL142	Adolescent/ Adult	273 x 220 x 135mm
EL141	Adult	280 x 240 x 140mm



P \* Adolescent only

S



### OASIS ELITE SUPINE HEAD REST

This Supine Head Rest is ideal for positioning the patient during eye surgery and other supine procedures. Ergonomically shaped to position the patient's head, while effectively protecting against the onset of pressure sores.

CODE	DESCRIPTION	DIMENSIONS
*EL152	Adolescent	275 x 230 x 65mm
EL151	Adult	275 x 237 x 90mm



P

\* Adolescent use only

S

L LL



### OASIS ELITE CLOSED HEAD RING

Unique gel & foam combination conforms comfortably to the patient's head, this head ring is suitable for use in many surgical procedures.

CODE	DESCRIPTION	DIMENSIONS
*EL011	Infant/Adolescent	Ø140 x 34mm
EL012	Adult	Ø200 x 50mm




P

\* Infant/Adolescent only

P

LL

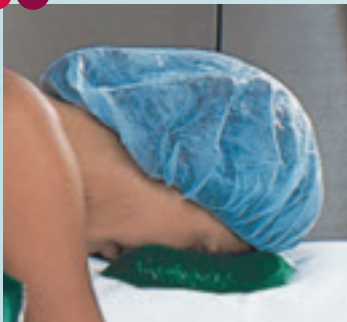

### OASIS ELITE OPEN HEAD RING

Combination of gel and foam provides stability and protection to the patient's face during surgery. An opening allows for intubation.

CODE	DESCRIPTION	DIMENSIONS
*EL021	Infant/Adolescent	Ø140 x 34mm
EL022	Adult	Ø200 x 54mm



P

\* Infant/Adolescent only

S L LL



P

### OASIS CLOSED HEAD RING

Available in a wide variety of size options, the Oasis Closed Head Ring protects the head, neck and ears during all types of surgery and is suitable for use in many procedures.

CODE	DESCRIPTION	DIMENSIONS
OA010	Neonate/New born	Ø90 x 20mm
OA011	Infant / Adolescent	Ø140 x 35mm
OA012	Adolescent/ Adult	Ø200 x 45mm



P LL



P

### OASIS OPEN HEAD RING

Soft, comfortable and durable, the Oasis Open Head Ring supports the patient's face, while allowing easy access for intubation during surgery.

CODE	DESCRIPTION	DIMENSIONS
OA020	Neonate/New born	Ø90 x 22mm
OA021	Infant / Adolescent	Ø140 x 35mm
OA022	Adolescent/ Adult	Ø200 x 50mm



S L LL



P

### OASIS FACE PAD

The Oasis Face Pad is ergonomically designed to protect the patient's forehead from pressure caused by endotracheal tubes.

CODE	DESCRIPTION	DIMENSIONS
OA140	Face Pad	165 x 80 x 8mm

S P LL



P


### AZURE ARM PROTECTOR

The Azure Arm Protector assumes the shape of the individual patient's arm, acting as a second layer of tissue and offering excellent protection against nerve damage.

CODE	DESCRIPTION	DIMENSIONS
AZ200	Infant/Adolescent/Adult	330 x 140 x 30mm
AZ201	Adolescent/ Adult	460 x 180 x 35mm




S P L LL

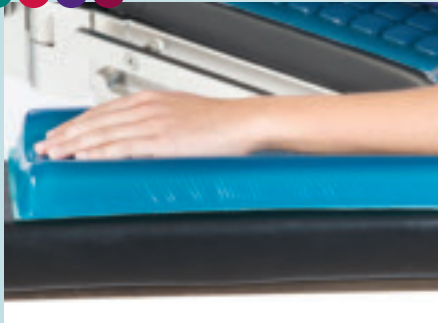

### OASIS ELITE CONTOURED ARM BOARD PAD

This anatomically shaped armboard pad is designed to provide added support and stability to the arm in a variety of procedures.

CODE	DESCRIPTION	DIMENSIONS
EL101	Adult	510 x 150 x 35mm
EL102	Adolescent/ Adult	395 X 140 X 40mm

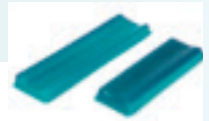

S P L LL



P

### OASIS ELITE ARMBOARD PAD

This flat armboard pad has a slight indentation to cradle the arm and provide effective pressure relief during various surgical procedures.

CODE	DESCRIPTION	DIMENSIONS
EL072	Adolescent/Adult	600 x 125 x 20mm
EL070	Infant/Adolescent/Adult	400 x 125 x 20mm

S P L LL



### OASIS CONTOURED ARMBEARD PAD

This contoured armboard pad has been designed for use on a standard table extension. Its contoured shape provides excellent protection as well as keeping the arm in place.

CODE	DESCRIPTION	DIMENSIONS
OA101	Armboard Pad	510 x 150 x 35mm



S P L LL



### OASIS ARMBEARD PADS

Oasis Armboard Pads provide the necessary protection whenever the arm is extended on an armboard. Suitable for all types of armboards.

CODE	DESCRIPTION	DIMENSIONS
OA070	Infant/Adolescent/Adult	395 x 115 x 10mm
OA071	Infant/Adolescent/Adult	600 x 200 x 10mm
OA072	Adolescent / Adult	600 x 125 x 10mm
OA073	Adolescent / Adult	605 x 427 x 10mm



P

S L LL




### OASIS ULNAR / BRACHIAL NERVE PROTECTORS

The Oasis Ulnar / Brachial Nerve Protectors are essential for providing protection to the sensitive areas of the elbows and arms.

CODE	DESCRIPTION	DIMENSIONS
OA060	Adolescent/ Adult	380 x 115 x 8mm
OA061	Adolescent/ Adult	460 x 210 x 8mm
OA062	Adolescent/ Adult	460 x 380 x 8mm

P \* Adolescent only

S

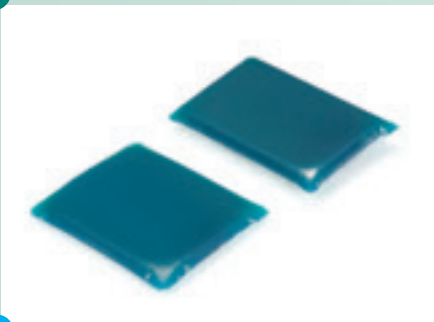

### OASIS ARM RETAINER PAD

The Oasis Arm Retainer Pad has been specially designed to fit perspex / acrylic arm supports. It can be easily fitted with Velcro attachments.

CODE	DIMENSIONS
OA100	170 x 110 x 18mm
OA102	157 x 128 x 15mm



P

S L



### AZURE SACRAL PROTECTOR

The Sacral area is an area of serious risk if left unprotected. The Azure Sacral protector has been designed to provide support in the supine and lithotomy positions by relieving pressure and focussing on peripheral support.

CODE	DESCRIPTION	DIMENSIONS
*AZ300	Adolescent/Adult	400 x 380 x 25mm
AZ301	Adult	450 x 445 x 30mm



P \* Adolescent only

S P LL



### AZURE UNIVERSAL POSITIONER

This versatile and lightweight product can be used to position and protect the patient in variety of surgical procedures.

CODE	DESCRIPTION	DIMENSIONS
AZ500	Adolescent/ Adult	425 x 155 x 60mm



P \* Adolescent only



P LL



### OASIS ELITE PRONE LATERAL POSITIONING SYSTEM

The Prone/Lateral Positioning System from Trulife is the result of extensive research into the needs of Operating Room personnel for patients during surgical procedures in the prone/lateral position.

Designed with both patient and surgical staff in mind, this system offers the ultimate pressure relieving protection and effective positioning all in one.



CODE	DESCRIPTION	DIMENSIONS
EL905	Oasis Elite Prone/Lateral Positioning System	675 x 500 x 100mm
<b>REPLACEMENT PARTS AVAILABLE:</b>		
EL906	Upper Left Section /Lower Right Section	275 x 250 x 100mm
EL907	Upper Right Section /Lower Left Section	275 x 250 x 100mm
EL908	Mid Left Section /Mid Right Section	125 x 250 x 100mm
EL912	Base Plate	500 x 500 x 6mm

P



### OASIS ELITE PRONE POSITIONER

The Prone Positioner gel and foam combination has been designed to provide effective positioning to the patient during surgical procedure in the prone position. For best protection, it is recommended to be used in conjunction with the Prone Head Rest.

CODE	DESCRIPTION	DIMENSIONS
EL903	Adult	610 x 530 x 130mm





P



### OASIS ELITE LARGE PRONE POSITIONER

The Large Prone Positioner eliminates pressure on the abdomen and facilitates easy ventilation and circulation during long cases in prone position, i.e. surgeries to the backs of legs and laminectomy procedure.

CODE	DESCRIPTION	DIMENSIONS
EL904	Adult	1115 x 520 x 140mm



LL



### OASIS ELITE LATERAL POSITIONER

This positioning device permits a safe, quick and easy transition from supine to lateral. The patient's lower shoulder fits into the integral channel to provide exceptional patient comfort and nerve protection. The Lateral Positioner is designed for use in hip replacement, shoulder, kidney and other procedures. The built-in headrest and auxiliary support assists in proper spinal alignment and surgical site access.

CODE	DESCRIPTION	DIMENSIONS
*EL900	Adolescent/ Adult	720 x 500 x 160mm



P

\* Adolescent only: Lateral Position

S

P



### OASIS ELITE DOME POSITIONER

This lightweight positioner can be used to position and protect the patient during various surgical procedures in all positions. It can be used in many different applications.

CODE	DESCRIPTION	DIMENSIONS
*EL210	Adolescent / Adult	350 x 150 x 75mm

P

\* Adolescent only: For Prone Position

S P L LL



### OASIS ELITE FLAT BOTTOM CHEST ROLLS

Chest Rolls provide excellent support and stability. The unique gel and foam combination means they are up to 75% lighter than comparative gel products, allowing for easy manipulation and handling by OR staff.

CODE	DESCRIPTION	DIMENSIONS
EL214	Adult	300 x 140 x 110
EL215	Adult	400 x 140 x 110
EL216	Adult	505 x 140 x 110
EL217	Adult	405 x 150 x 150
EL218	Adult	510 x 150 x 150

S P L LL



### OASIS FLAT BOTTOM CHEST ROLLS

These 100% silicone gel chest rolls provide excellent positioning and stability and are available in a wide variety of sizes.

CODE	DESCRIPTION	DIMENSIONS
*OA208	Infant	150 x 80 x 50mm
*OA209	Infant	200 x 80 x 50mm
*OA210	Infant / Adolescent	245 x 80 x 50mm
OA211	Adult	305 x 95 x 75mm
OA212	Adult	405 x 95 x 75mm
OA213	Adult	505 x 92 x 75mm



P \*Specific for Paediatrics

S P L LL



### OASIS PATIENT POSITIONING BELT

The Oasis Patient Positioning Belt ensures patient safety by providing stability and tissue protection on the Operating Table. By using the clip-lock attachment the belt can be easily adjusted to fit all operating tables. In case of emergency, there is an easy release clip lock that can be removed quickly.

CODE	DIMENSIONS
OA110	505 x 85 x 8mm

S P L LL



### OASIS EXTREMITY BELT

The Oasis Extremity Belt is used in many procedures for extra security in maintaining the position of the extremities on the operating table.

#### CODE


OA113

#### DIMENSIONS

315 x 45 x 10mm

P

S P L LL

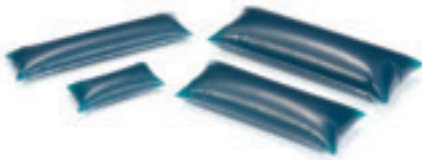

### OASIS UNIVERSAL PATIENT POSITIONERS

These 100% silicone positioners are suitable for safe and effective positioning during all surgical procedures.

#### CODE

#### DESCRIPTION

#### DIMENSIONS

OA050	Infant/Adolescent/Adult	150 x 60 x 40mm
OA051	Infant/Adolescent/Adult	330 x 70 x 50mm
OA052	Adolescent / Adult	290 x 100 x 60mm
OA053	Adolescent / Adult	345 x 120 x 70mm

P

S



### OASIS PERINEAL POST PAD

The Oasis Perineal Post Pad reduces pressure on the patient in the groin area and is attached quickly and easily with Velcro. Its universal shape and size allows it to be used on most operating room fracture tables.

#### CODE

OA180

#### DIMENSIONS

300 x 260 x 10mm


S P LL



**AZURE LEG PROTECTOR**

The Azure Leg Protector provides overall protection to the heel and leg area. Slightly raised behind the knee, the leg protector offers protection to the popliteal nerve and conforms superbly around the leg. Overall positioning, protection and alignment are the key features of this unique product.

CODE	DESCRIPTION	DIMENSIONS
*AZ400	Adolescent / Adult	515 x 180 x 40mm

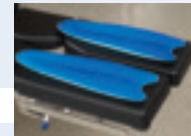

P \* Adolescent only

L



**AZURE CRUTCH STIRRUP PADS**

These Crutch Stirrup Pads were designed for use with various types of legholders to protect the nerves and blood vessels in the leg during lithotomy procedures. Lightweight and flexible, the dual layer Azure pads come in pairs and protect against skin shear and nerve injury. Supplied in pairs.

CODE	DIMENSIONS
AZ410	370 x 205 x 25mm

LL



**OASIS ELITE LATERAL LEG POSITIONER**

This product is used to position the legs in a lateral position for kidney surgery and hip surgery. In addition to comfort, it facilitates optimal surgical positioning. This positioner also allows for knee and hip flexion. To use: Place patient on side, place upper limb on top (gel part of the product), and place lower limb underneath ledge.

CODE	DESCRIPTION	DIMENSIONS
EL800	Adult	650mm x 400mm x 230mm

P



**OASIS ELITE KNEE PAD**

The Knee pad is used for various procedures in the prone position to protect the knees against pressure damage and the onset of pressure sores.

CODE	DESCRIPTION	DIMENSIONS
EL401	Adult	510 x 150 x 35mm
*EL402	Adolescent/ Adult	400 x 150 x 35mm



P \* Adolescent only

L



**OASIS CRUTCH STIRRUP PADS**

The Oasis Crutch Stirrup Pads are designed to fit most types of stirrups. Supplied in pairs, they come in a variety of sizes.

CODE	DESCRIPTION	DIMENSIONS
OA090	Adolescent / Adult	320 x 220 x 10mm
OA091	Adolescent / Adult	395 x 245 x 10mm
OA092	Adolescent / Adult	520 x 300 x 10mm



S



### AZURE HEEL PROTECTOR

The heel area is one of extreme risk during any surgical procedure and special attention should be given to avoid any unnecessary damage. These lightweight protectors provide heel protection as well as stability and support. Supplied in pairs.

CODE	DESCRIPTION	DIMENSIONS
AZ401	Infant/Adolescent/ Adult	190 x 175 x 30mm



P

S



### OASIS ELITE HEEL PADS

The Oasis Elite Heel Pads provide support and maximum protection for the heels, even during the longest surgical procedures. They support the Achilles and effectively remove pressure from the heels. Supplied in pairs, the products are lightweight and easy to use.

CODE	DESCRIPTION	DIMENSIONS
*EL130	Adolescent / Adult	180 x 100 x 70mm



P

\* Adolescent only

**S**



**OASIS HEEL PADS**

These 100% silicone gel pads are suitable for use in any supine procedure where the heels are at risk of skin breakdown and damage.

CODE	DESCRIPTION	DIMENSIONS
*OA130	Adolescent/Adult	180 x 100 x 70mm

**P** \* Adolescent only

**S**



**OASIS HEEL CUPS**

Oasis Heel Cups are designed to provide pressure relief for patient's heels. They are suitable for use during and after most surgical procedures and can be used also for patients who are confined to bed or are in a day chair.

CODE	DESCRIPTION	DIMENSIONS
OA131	Neonate/ Infant/ Adolescent/Adult	105 x 90 x 43mm



**P**

**S L LL**



**OASIS HEEL & ANKLE PROTECTOR**

These unique pads are designed to fit around the heel and ankle like a boot with Velcro attachment. They provide protective padding for the heel when the patient is in the supine or lateral positions. Can also be used as a protector for the ankle and foot when using conventional ankle strap stirrups and lithotomy poles.

CODE	DESCRIPTION	DIMENSIONS
*OA200	Adolescent/ Adult	250 x 120 x 10mm



**P** \* Adolescent only: For Lateral



L



### OASIS BOOT STIRRUP PAD

These pads provide excellent pressure relief and protection to the calf and heel and are designed for use on most types of boot stirrups. The pads are secured to the boot stirrup via Velcro.

CODE	DESCRIPTION	DIMENSIONS
*OA160	Adult	605 x 140 x 5mm

\* Also suitable for Fowler's position

P



### OASIS RELTON FRAME PADS

Designed to fit over the four support pads of the Relton frame and provide excellent protection to the patient when positioned for spinal surgery.

CODE	DESCRIPTION	DIMENSIONS
OA120	Adult	190 x 170 x 18mm



P



### OASIS WILSON FRAME PADS


Wilson Frame Pads are designed to provide pressure relief for patients positioned on a Wilson Frame, used to maintain patients in a flexed spine position.

CODE	DESCRIPTION	DIMENSIONS
OA221	Adult	735 x 255 x 10mm






INCUBATOR PAD  
OA300

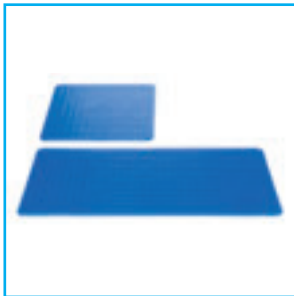

AZURE PROFILE PADS  
AZ610 / AZ611




AZURE TABLE PADS  
AZ600 / AZ601

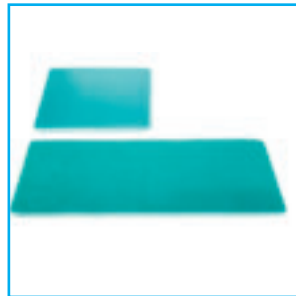

OASIS ELITE TABLE PADS  
EL030 / EL031



OASIS STANDARD OPERATING  
TABLE PADS  
OA030 / OA031 / OA032

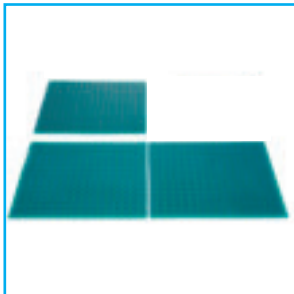


OASIS PROFILE PAD  
OA041 / OA042



OASIS 3 PIECE OPERATING  
TABLE PAD  
OA036 / OA037 / OA038  
/ OA039



OASIS FULL LENGTH  
PAEDIATRIC PAD  
OA033

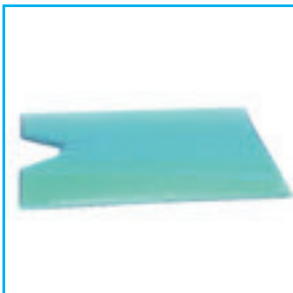

OASIS PERINEAL TABLE PAD  
OA040



AZURE HEAD PROTECTOR  
AZ100



OASIS ELITE PRONE  
HEAD REST  
EL142



OASIS ELITE SUPINE  
HEAD REST  
EL152



OASIS ELITE CLOSED  
HEAD RING  
EL011




OASIS ELITE OPEN  
HEAD RING  
EL021



OASIS CLOSED  
HEAD RING  
OA010 / OA011



OASIS OPEN HEAD RING  
OA020 / OA021

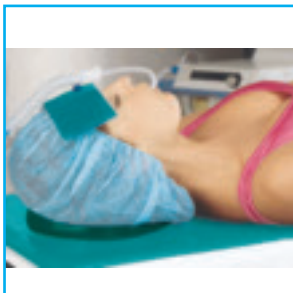

OASIS FACE PAD  
OA140



AZURE ARM PROTECTOR  
AZ200



OASIS ELITE ARMBEARD PAD  
EL070



OASIS ARMBEARD PADS  
OA070 / OA071 / OA072 / OA073



OASIS ULNAR / BRACHIAL  
NERVE PROTECTORS  
OA060 / OA061 / OA062



OASIS ARM RETAINER PAD  
OA100 / OA102



AZURE SACRAL PROTECTOR  
AZ300



AZURE UNIVERSAL POSITIONER  
AZ500



OASIS ELITE LATERAL POSITIONER EL900




OASIS ELITE DOME POSITIONER EL210



OASIS FLAT BOTTOM CHEST ROLLS OA208 / OA209 / OA210



OASIS EXTREMITY BELT OA113

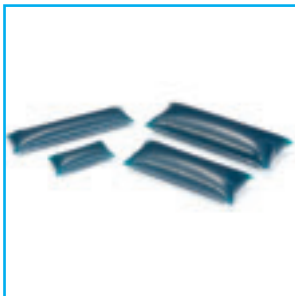

OASIS UNIVERSAL PATIENT POSITIONERS OA050 / OA051 / OA052 / OA053



AZURE LEG PROTECTOR AZ400



OASIS ELITE KNEE PAD EL402



AZURE HEEL PROTECTOR AZ401



OASIS ELITE HEEL PADS EL130



OASIS HEEL PADS OA130



OASIS HEEL CUPS OA131



OASIS HEEL & ANKLE PROTECTOR OA200





Technical Guide

**OEACARD12**

Pressurecare for Operating Room, Decubitus and Nerve Injury Prophylaxis

<ul style="list-style-type: none"> <li>• Original Gel Solution</li> <li>• 100% Silicone</li> <li>• Similar Texture to Human Skin</li> </ul>	<ul style="list-style-type: none"> <li>• Gel &amp; Foam</li> <li>• Pre-Contoured</li> <li>• 50% Lighter than Oasis</li> </ul>	<ul style="list-style-type: none"> <li>• Anatomical Design</li> <li>• Dual-Layer of Silicone</li> <li>• 20% Lighter than Oasis</li> </ul>

**Trulife**

- Medical Grade Silicone
- Easy to Clean
- Latex Free
- Biocompatible
- Reusable
- MRI Compatible
- X-ray Translucent
- Most Extensive Range

www.trulife.com

Distributor: \_\_\_\_\_

Operating Room Poster

**Trulife**

**PROTECTION FOR ALL SURGICAL PROCEDURES**

<p><b>SUPINE POSITION</b></p> <ul style="list-style-type: none"> <li>• Administration of General Anesthesia</li> <li>• Abdominal Surgery</li> <li>• Open-Heart Surgery</li> <li>• Tummy and Back</li> <li>• Head or Neck</li> <li>• Hip and Leg</li> <li>• Orthopedics</li> </ul>
<p><b>PRONE POSITION</b></p> <ul style="list-style-type: none"> <li>• Endotracheal Tube</li> <li>• Spine and the back of the legs</li> </ul>
<p><b>LATERAL POSITION</b></p> <ul style="list-style-type: none"> <li>• Surgery of the Chest or Lung</li> <li>• Surgery on the Hip</li> <li>• Hip Surgery</li> </ul>
<p><b>LITHOTOMY POSITION</b></p> <ul style="list-style-type: none"> <li>• Urological Surgery</li> <li>• Gynecological Surgery</li> </ul>

www.trulife.com

Positioning Guide

**SUPINE POSITION**

INCLUDES TRENDELENBURG & REVERSE TRENDELENBURG

The patient is on their back with face toward ceiling, legs are uncrossed and arms are at the side or on arm boards.



BODY AREA	PRODUCT DESCRIPTION	CODE
HEAD & NECK	Head Protector	42700
	Child/Head Ring	42701 / 42702 / 42703 / 42704
	Support/Head Rest Face Pad	42705
ARM & ELBOW	Arm Protector	42706 / 42707
	Armboard Pad	42708 / 42709 / 42710 / 42711
	Ulnar / Brachial Protector	42712
	Arm Restraint Pad	42713 / 42714
BODY	Torso Pads	42715 / 42716 / 42717 / 42718 / 42719 / 42720 / 42721 / 42722 / 42723 / 42724
	Spinal Protector	42725
	Universal Protector	42726
	Spine Protector	42727
	Head Protection	42728 / 42729 / 42730
	Positioning Belt	42731
	Extremity Rest	42732
	Reinforced Floor Pad	42733
LEG & KNEE	Leg Protector	42734
ARM, NECK & FOOT	Head Protector	42735
	Head Pad	42736 / 42737
	Head Cradle	42738
	Head & Neck Protector	42739
	Neck/Head Pad	42740
	Oral Rest	42741 / 42742 / 42743 / 42744

www.trulife.com






BODY AREA	PRODUCT DESCRIPTION	CODES
<b>HEAD &amp; NECK</b>	Head Protector Closed Head Rings  Supine Head Rest Face Pad	AZ100 EL011 / 12 OA010 / 11 / 12 EL151 / 52 OA140
<b>ARM &amp; ELBOW</b>	Arm Protectors Arm/Kneeboard Pads  Ulnar / Brachial Protector Arm Retainer Pad	AZ200 / 201 EL070 / 72 / 101 / 102 OA101 / 070 / 071 / 072 / 073 OA060 / 061 / 062 OA100 / 102
<b>BODY</b>	Table Pads  Sacral Protector Universal Positioner Dome Positioner Patient Positioners Positioning Belt Extremity Belt Perineal Post Pad Chest Rolls	AZ 600 / 601 / 610 / 611 EL030 / 31 OA030 / 31 / 32 / 35 / 36 / 37 / 38 / 39 / 40 / OA041 / 42 AZ300 / 301 AZ500 EL210 OA050 / 51 / 52 / 53 OA110 OA113 OA180 EL214 / 215 / 216 / 217 / 218 OA211 / 212 / 213
<b>LEG &amp; KNEE</b>	Leg Protector	AZ400
<b>ANKLE, HEEL &amp; FOOT</b>	Heel Protector Heel Pads Heel Cups Heel & Ankle Protector	AZ401 EL130 / OA130 OA131 OA200

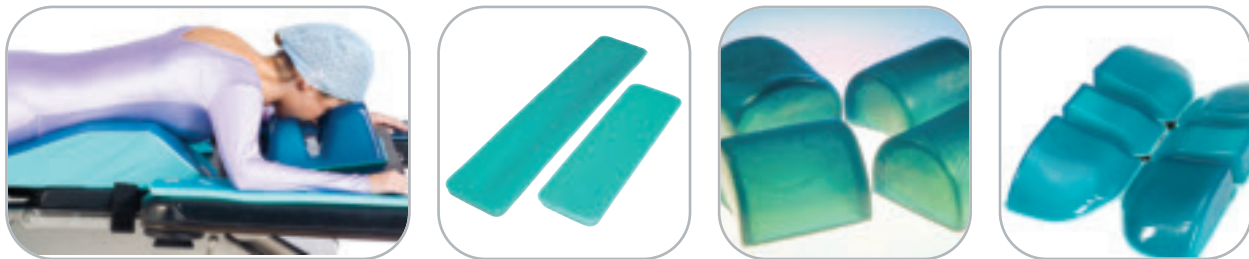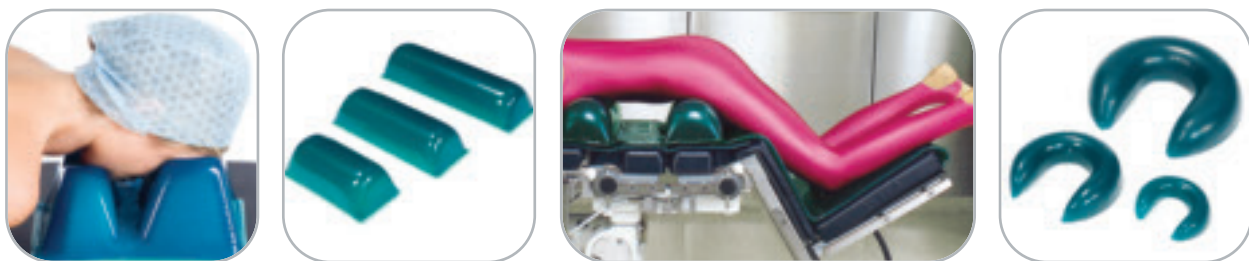

BODY AREA	PRODUCT DESCRIPTION	CODES
<b>HEAD &amp; NECK</b>	Head Protector Open Head Rings	AZ100 EL021 / 22 OA020 / 21 / 22
	Prone Headrest	EL141 / 142
<b>ARM &amp; ELBOW</b>	Arm Protectors Armboard Pads	AZ200 / 201 EL070 / 72 / 101 / 102 OA101 / 070 / 071 / 072 / 073
<b>BODY</b>	Table Pads	AZ 600 / 601 / 610 / 611 EL030 / 31 OA030 / 31 / 32 / 35 / 36 / 37 / 38 / 39 / 40 / OA041 / 42
	Prone Positioning System Prone Positioner Large Prone Positioner Universal Positioner Chest Rolls	EL905 / 906 / 907 / 908 / 912 EL903 EL904 AZ500
	Dome Positioner Patient Positioners Extremity Belt Positioning Belt	EL214 / 215 / 216 / 217 / 218 OA211 / 12 / 13 EL210 OA050 / 51 / 52 / 53 OA113 OA110
<b>LEG &amp; KNEE</b>	Arm / Knee Pad Leg Protector	EL401 / 402 AZ400
<b>ANKLE, HEEL &amp; FOOT</b>	Dome Positioner	EL210
<b>SPECIALITY PAD</b>	Relton Frame Pads Wilson Frame Pads	OA120 OA221



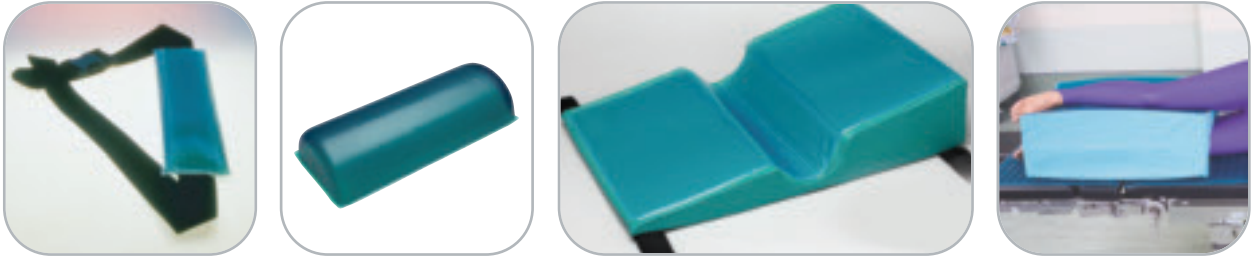

BODY AREA	PRODUCT DESCRIPTION	CODES
<b>HEAD &amp; NECK</b>	Open Head Ring Head Protector Closed Head Rings Face Pad	EL021 / 022 OA020 / 021 / 022 AZ100 EL011 / 12 OA010 / 11 / 12 OA140
<b>ARM &amp; ELBOW</b>	Arm/Knee Protectors Armboard Pads Ulnar / Brachial Protector	AZ200 / 201 EL070 / 72 / 101 / 102 OA101 / 070 / 071 / 072 / 073 OA060 / 061 / 062
<b>BODY</b>	Table Pads  Lateral Positioner Prone Positioning System Universal Positioner Patient Positioners Positioning Belt Extremity Belt Flat Bottom Chest Rolls	AZ600 / 601 / 610 / 11 EL030 / 31 OA030 / 31 / 32 / 33 / 35 / 36 / 37 / 38 / 39 / 40 / OA041 / 42 EL900 EL905 / 906 / 907 / 908 / 912 AZ500 OA050 / 51 / 52 / 53 OA110 OA113 EL214 / 215 / 216 / 217 / 218 OA211 / 212 / 213
<b>LEG</b>	Azure Leg Protector	AZ400
<b>ANKLE, HEEL &amp; FOOT</b>	Heel & Ankle Protector Oasis ELite Lateral Leg Positioner	OA200 EL800





# LITHOTOMY

BODY AREA	PRODUCT DESCRIPTION	CODES
<b>HEAD &amp; NECK</b>	Head Protector Closed Head Rings	AZ100 EL011 / 12 OA010 / 11 / 12
	Face Pad	OA140
<b>ARM &amp; ELBOW</b>	Arm Protectors Arm/Legboard Pads	AZ200 / 201 EL070 / 71 / 72 / 73 / 101 / 102 OA101 / 070 / 071 / 072 / 073
	Ulnar / Brachial Protector Arm Retainer Pad	OA060 / 061 / 062 OA100 / 102
<b>BODY</b>	Table Pads	AZ600 / 601 / 610 / 11 EL030 / 31 OA030 / 31 / 32 / 33 / 35 / 36 / 37 / 38 / 39 / 40 / OA041 / 42
	Incubator Pad Sacral Protector Patient Positioners Positioning Belt Extremity Belt Flat Bottom Chest Rolls	OA300 AZ300 / 301 OA050 / 51 / 52 / 53 OA110 OA113 EL214 / 215 / 216 / 217 / 218 OA208 / 209 / 210 / 211 / 212 / 213
<b>LEG &amp; KNEE</b>	Crutch Stirrup Pad	AZ410 OA090 / 91 / 92
<b>ANKLE, HEEL &amp; FOOT</b>	Heel & Ankle Protector Boot Stirrup Pad	OA200 OA160



WITHOUT PROTECTION





HEAD

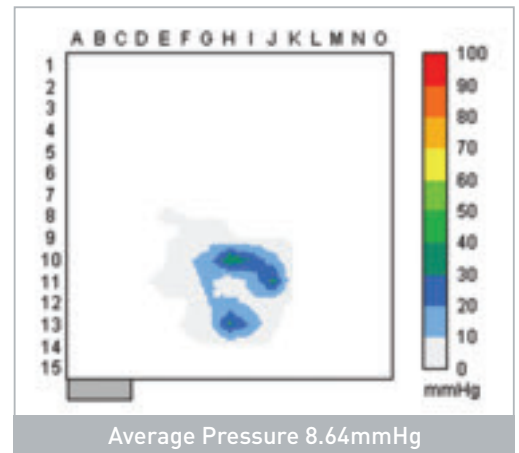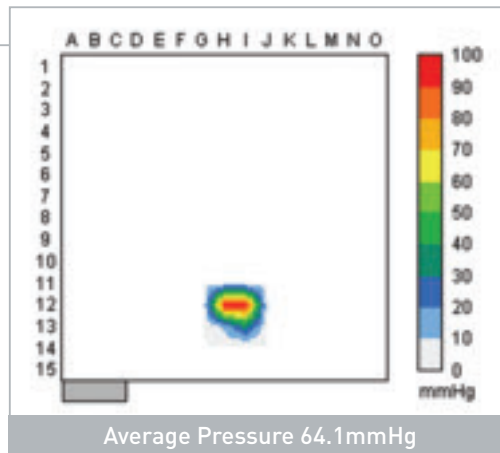







ARMS

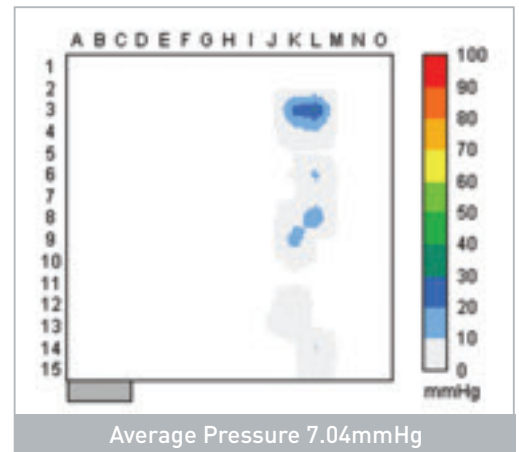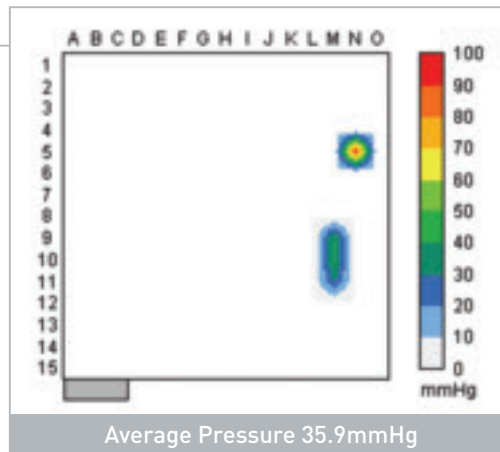

HEELS

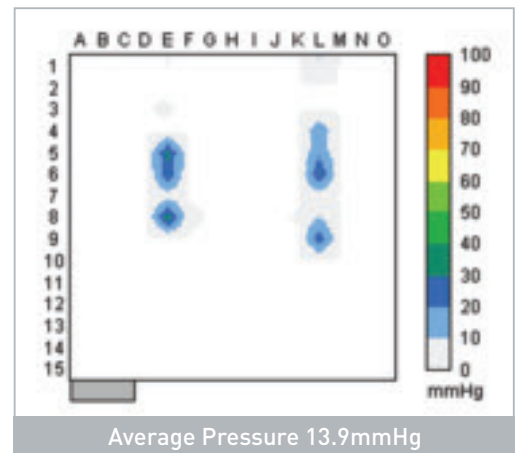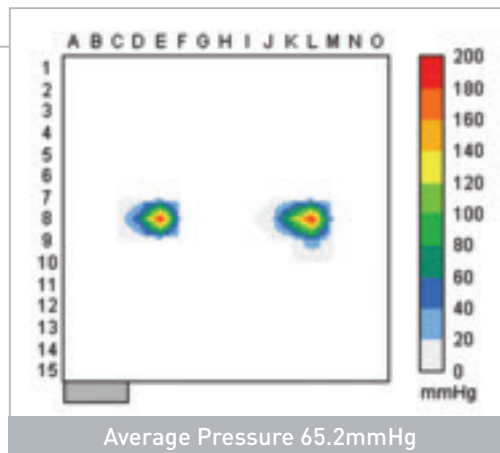

Tested in accordance with protocol for interface pressure management Ref: FSAT.0017

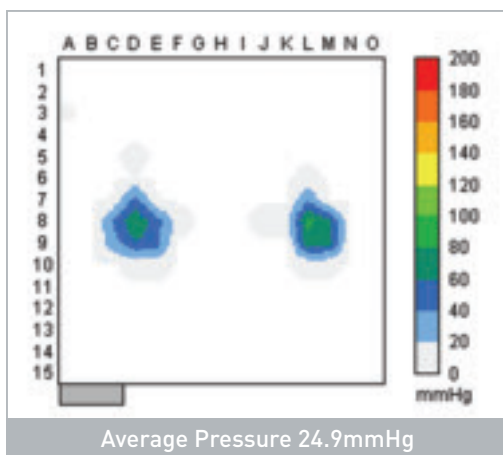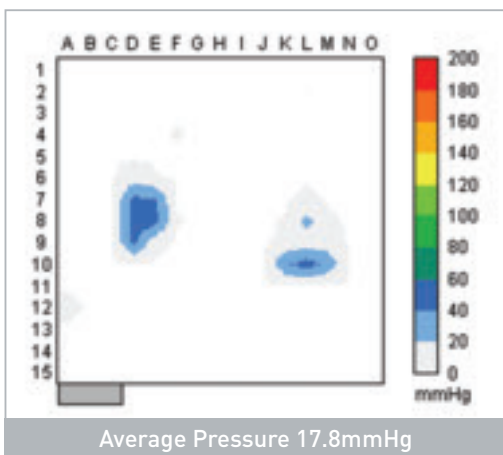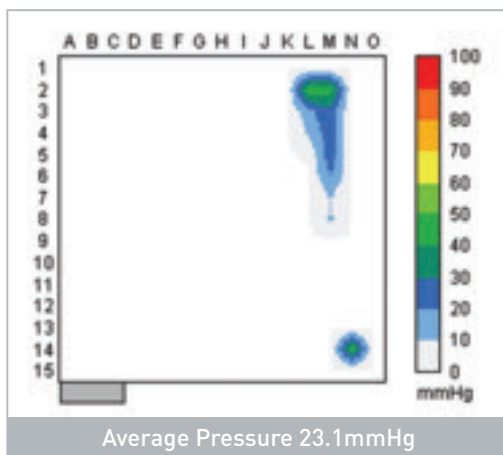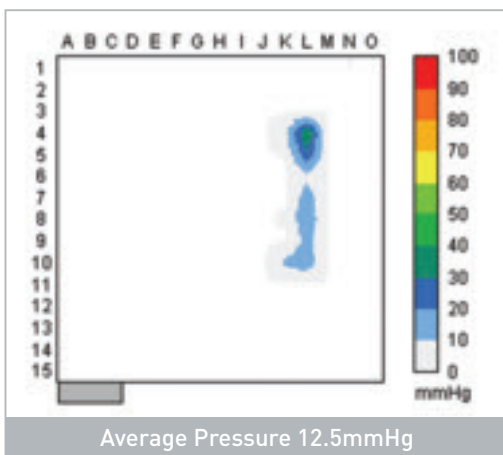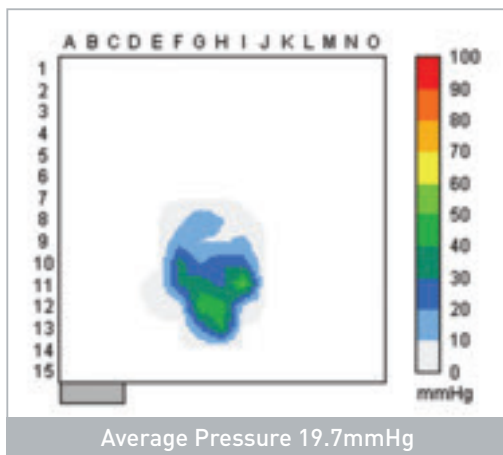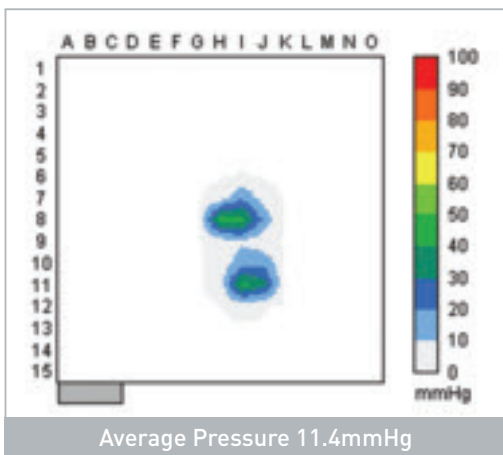

“ Any positioning device used to reduce pressure on patient's body parts and prevent tissue damage should have the documented ability to reduce capillary interface pressures to 32mmHg or less (5) ”

<sup>5</sup> Hendrick, J. & Thomson et al. "Pressure reduction products: making appropriate choices" Journal of Enterostomal Nursing 20 (Nov/Dec 1993)



Each and every product in the Trulife Pressurecare ranges have been subjected to the most rigorous of durability tests to ensure that they will withstand normal day to day handling in even the busiest surgical environment. In the design phase, tests are conducted in relation to the every day effects of compression, rolling, pulling and tearing.

The Trulife Pressurecare ranges consist of the most extensive range of gel pads and positioners available and can be used for a variety of applications in practically every surgical procedure.

The products are manufactured under strict ISO: 13485:2003 standards and carry the CE mark.

#### HOW CAN THE PRODUCTS BE CLEANED?

All products can be easily cleaned using standard OR environment detergents. It is not advised to soak the products over long periods. The products are not suitable for use in conjunction with an autoclave.

#### CAN THE PRODUCTS BE USED IN DIRECT CONTACT WITH THE PATIENT'S SKIN?

All products are latex-free and non-allergenic. The gel used in the products is a cured silicone and does not flow. It is an inert material and does not support the growth of bacteria.

#### ARE YOUR PADS X-RAY TRANSLUCENT?

Our pads are x-ray translucent, however, higher radiation levels may be needed.

#### CAN THE PRODUCTS BE HEATED OR COOLED?

The products can be cooled to -12°C without any adverse effects. As adhesion of velcro can be weakened by low temperatures, products with velcro should be allowed to return to room temperature prior to use.

The products can also be heated to a maximum of 40°C if required and can be safely used in conjunction with heated blankets.

#### CAN THE PADS BE USED WITH MRI EQUIPMENT?

The materials used in our Oasis/Elite/Fusion and Azure products are a combination of PU film and silicone. They contain no ferrous material and so are considered MRI friendly.

#### WHAT IF THE PRODUCT GETS CUT?

While a repair kit is available, in the event of the product being cut, it is highly recommended that the product be discarded to avoid the risk of infection.

#### DO THE PRODUCTS CARRY A GUARANTEE?

All products are guaranteed against manufacturing defects for a period of 2 years.

#### PLEASE NOTE:

- Due to our manufacturing process a slight variation in colour may occur between individual products.
- Product dependent a tolerance of +/- 10mm is applied to all product dimensions.
- The position of the model in certain images is intended to display the products only and may not be surgically accurate in all cases.
- We offer a guideline to help select products for different surgical positions. Please refer to good clinical practices when selecting products.



CODE	DESCRIPTION	DIMENSIONS (mm)	PAGE	CODE	DESCRIPTION	DIMENSIONS (mm)	PAGE
<b>TABLE PAD PROTECTION</b>				<b>BODY PROTECTION</b>			
AZ610	Azure Universal Table Pad	520 x 520 x 10	4	AZ300	Azure Sacral Protector Small	400 x 380 x 25	12
AZ611	Azure ¾ Length Table Pad	1150 x 520 x 10	4	AZ301	Azure Sacral Protector Large	450 x 445 x 30	12
AZ600	Operating Table Pad	520 x 520 x 10	4	AZ500	Azure Universal Positioner	425 x 155 x 60	12
AZ601	Operating Table Pad	1150 x 520 x 10	4	EL903	Oasis Elite Prone Positioner	610 x 530 x 130	13
EL030	Oasis Elite Universal Table Pad	520 x 520 x 30	4	EL904	Oasis Elite Large Prone Positioner	1115 x 520 x 140	14
EL031	Oasis Elite ¾ Length Table Pad	1150 x 520 x 20	4	EL905	Oasis Elite Prone Positioning System	675 x 500 x 100	13
OA030	Oasis Universal Table Pad	520 x 520 x 10	4	EL906	Upper Left Section/Lower Right Section	275 x 250 x 100	13
OA031	Oasis ¾ Length Table Pad	1150 x 520 x 10	4	EL907	Upper Right Section/Lower Left Section	275 x 250 x 100	13
OA032	Oasis Full Length Table Pad	1800 x 520 x 10	4	EL908	Mid Left Section/Mid Right Section	125 x 250 x 100	13
OA041	Oasis Universal Profile Pad	520 x 520 x 12	5	EL912	Base Plate	500 x 500 x 6	13
OA042	Oasis ¾ Length Profile Pad	1145 x 520 x 12	5	EL900	Oasis Elite Lateral Positioner	720 x 500 x 160	14
OA036	Oasis Head Section	225 x 500 x 10	5	EL210	Oasis Elite Dome Positioner	350 x 150 x 75	14
OA037	Oasis Torso Section	990 x 500 x 10	5	EL214	Oasis Elite Flat Bottom Chest Rolls	300 x 140 x 110	15
OA038	Oasis Foot Section	500 x 500 x 10	5	EL215	Oasis Elite Flat Bottom Chest Rolls	400 x 140 x 110	15
OA039	Oasis 3 Piece Set		5	EL216	Oasis Elite Flat Bottom Chest Rolls	505 x 140 x 110	15
OA033	Oasis Paediatric Full Length Pad	605 x 335 x 10	5	EL217	Oasis Elite Flat Bottom Chest Rolls	405 x 150 x 150	15
OA035	Oasis Hip Pad	520 x 520 x 10	6	EL218	Oasis Elite Flat Bottom Chest Rolls	510 x 150 x 150	15
OA040	Oasis Perineal Table Pads	520 x 520 x 10	6	OA208	Oasis Flat Bottom Chest Rolls	150 x 80 x 50	15
OA300	Incubator Pad	620 x 340 x 12	6	OA209	Oasis Flat Bottom Chest Rolls	200 x 80 x 50	15
<b>HEAD &amp; NECK PROTECTION</b>				<b>LEG &amp; KNEE PROTECTION</b>			
AZ100	Azure Head Protector	260 x 250 x 50	7	AZ400	Azure Leg Protector	515 x 180 x 40	17
EL142	Oasis Elite Prone Head Rest Medium	273 x 220 x 135	7	AZ410	Azure Crutch Stirrup Pads	370 x 205 x 25	17
EL141	Oasis Elite Prone Head Rest Large	280 x 240 x 140	7	EL800	Oasis Elite Lateral Leg Positioner	650 x 400 x 230	17
EL152	Oasis Elite Supine Head Rest Adolescent	275x 230 x 65	8	EL401	Oasis Elite Knee Pad	510 x 150 x 35	17
EL151	Oasis Elite Supine Head Rest Adult	275 x 237 x 90	8	EL402	Oasis Elite Knee Pad	400 x 150 x 35	17
EL011	Oasis Elite Closed Head Ring Adolescent	Ø140 x 34	8	OA090	Oasis Crutch Stirrup Pads	320 x 220 x 10	17
EL012	Oasis Elite Closed Head Ring Adult	Ø200 x 50	8	OA091	Oasis Crutch Stirrup Pads	395 x 245 x 10	17
EL021	Oasis Elite Open Head Ring Adolescent	Ø140 x 34	8	OA092	Oasis Crutch Stirrup Pads	520 x 300 x 10	17
EL022	Oasis Elite Open Head Ring Adult	Ø 200 x 54	8	<b>ANKLE, HEEL &amp; FOOT PROTECTION</b>			
OA010	Oasis Closed Head Ring Paediatric	Ø90 x 20	9	AZ401	Azure Heel Protector	190 x 175 x 30	18
OA011	Oasis Closed Head Ring Adolescent	Ø140 x 35	9	EL130	Oasis Elite Heel Pads	180 x 100 x 70	18
OA012	Oasis Closed Head Ring Adult	Ø 200 x 45	9	OA130	Oasis Heel Pads	180 x 100 x 70	19
OA020	Oasis Open Head Ring Paediatric	Ø90 x 22	9	OA131	Oasis Heel Cups	105 x 90 x 43	19
OA021	Oasis Open Head Ring Adolescent	Ø140 x 35	9	OA200	Oasis Heel & Ankle Protector	250 x 120 x 10	19
OA022	Oasis Open Head Ring Adult	Ø200 x 50	9	OA160	Oasis Boot Stirrup Pad	605 x 140 x 5	20
OA140	Oasis Face Pad	165 x 80 x 8	9	<b>SPECIALTY PAD PROTECTION</b>			
<b>ARM &amp; ELBOW PROTECTION</b>				OA120	Oasis Relton Frame Pads	190 x 170 x 18	20
AZ200	Azure Arm Protector Short	330 x 140 x 30	10	OA221	Oasis Wilson Frame Pads	735 x 255 x 10	20
AZ201	Azure Arm Protector Long	460 x 180 x 35	10	<b>CONCLUSION</b>			
EL101	Oasis Elite Contoured Armboard Pad	510 x 150 x 35	10	Paediatric Products			21
EL102	Oasis Elite Contoured Armboard Pad	395 x 140 x 40	10	Accessories			25
EL072	Oasis Elite Armboard Pad Long	600 x 125 x 20	10	Positioning Guide			26
EL070	Oasis Elite Armboard Pad Short	400 x 125 x 20	10	Interface Pressure Maps			30
OA101	Oasis Contoured Armboard Pad	510 x 150 x 35	11	General Use			32
OA070	Oasis Armboard Pads	395 x 115 x 10	11				
OA071	Oasis Armboard Pads	600 x 200 x 10	11				
OA072	Oasis Armboard Pads	600 x 125 x 10	11				
OA073	Oasis Armboard Pads	605 x 427 x 10	11				
OA060	Oasis Ulnar/Brachial Nerve Protectors (S)	380 x 115 x 8	11				
OA061	Oasis Ulnar/ Brachial Nerve Protectors (L)	460 x 210 x 8	11				
OA062	Full Length	460 x 380 x 8	11				
OA100	Oasis Arm Retainer Pad	170 x 110 x 18	11				
OA102	Oasis Arm Retainer Pad	157 x 128 x 15	11				





ENQUIRIES	ADDRESS	TELEPHONE	FAX
UK:	Trulife, 41 Amos Road, Meadowhall , Sheffield, S9 1BX, UK	+44 114 26 18100	+44 1142610074
International:	Trulife, 3013 Lake Drive, Citywest Business Campus, Dublin 24, Ireland	+353 1 4511755	+353 1 4660015
Ireland:	Trulife, 3013 Lake Drive, Citywest Business Campus, Dublin 24, Ireland	+353 1 4511755	+353 1 4660015
USA:	Trulife, 2010 East High St., P.O. Box 89, Jackson, MI 49203, USA	+1 800 788 2267	+1 517 789 3299
Canada:	Trulife, 39 East Davis St, Trenton, Ontario, K8V 4K8, Canada	+1 800 267 2812	+1 613 392 4139

